

National Institute for Occupational Safety and Health  
 National Personal Protective Technology Laboratory  
 Technical Evaluation Branch  
 626 Cochran Mill Road  
 Pittsburgh, Pennsylvania 15236



Date: August 12, 2009

NOTE: The Technical Evaluation Branch maintains an updated index of current procedures.

Total Inward Leakage Test for Half-mask Air-purifying Particulate Respirators

1. Purpose

1.1. This procedure establishes test method for quantifying the Total Inward Leakage (TIL) for half-mask respirators under 42CFR Part84, Subpart K, including those having elastomeric and filtering facepieces. The purpose of the test is to (1) ensure that a half-mask respirators are capable of providing a minimum level of protection as measured by total inward leakage, over the range of facial sizes for which the facepieces were designed to fit and (2) ensure that the User’s Instructions describe how to achieve that fit. The respirators will be tested on human subjects, having facial sizes designated by the respirator manufacturer for the specific facepiece, from a NIOSH panel having facial sizes and shapes that approximate the distribution of sizes and shapes of the working population of the United States (see appendix A).

2. General

2.1. This procedure describes the TIL test for respirators using a Condensation Nuclei Counter (CNC) in an ambient atmosphere, in sufficient detail that a person knowledgeable in fit testing procedures can conduct the test, determine the level of protection afforded by the respirator and make a pass/fail determination based on specific requirements.

3. Equipment/ Materials

3.1. The following is a list of equipment necessary to conduct the test:

Approvals:	1 <sup>st</sup> Level	2 <sup>nd</sup> Level	3 <sup>rd</sup> Level	4 <sup>th</sup> Level



- 3.1.1. PortaCount<sup>®</sup> Pro+, CNC fit test apparatus and associated facepiece adapters and calibration equipment



- 3.1.2. One or more TSI particle generators.
- 3.1.3. Thirty-five of each size and style of non-cleanable respirator to be evaluated or three of each size and style of cleanable respirators.

### 3.2. Test Subjects

- 3.2.1. All test subjects shall meeting the requirements of NIOSH Human Test Subject Review Board (HSRB) approved protocol. Refer to HSRB-04-NPPTL-02XP “, and having facial dimensions falling within the requirements of the NIOSH bivariate test panel for the size of the respirator to be tested. See Appendix A.
- 3.2.2. All test subjects shall also have facial characteristics which result in being included within the Principal Components Analysis Panel, which excludes extreme facial features. See Appendix B.
- 3.2.3. For the purpose of measuring the TIL, 35 test subjects will be chosen for a single size or multiple size respirator designed to fit the entire population, or 15 or more test subjects for a single size or multiple size respirator intended to fit one or more specific subpopulations of respirator users matching the respirator sizing as described in instructions for use accompanying the respirator.

### 4. Testing Requirements and conditions

- 4.1. At a minimum, all measuring equipment utilized for this testing must have been calibrated within the preceding 12 months using a method traceable to the National Institute of Standards and Technology (NIST), or more frequently if specified by the equipment manufacturer. Equipment calibration records shall be available for examination at each testing facility. Prior to beginning any testing, a statement that all test equipment is within calibration shall be attested by the lab technician on

each test report.

- 4.2. Prior to conducting the test, the Users Instructions provided by the manufacturer shall be reviewed to verify that the instructions for facepiece size selection are easily understood, easily followed, and practical. Test subjects will familiarize themselves with the manufacturer's selection, donning and fitting procedures for the respirator.
  - 4.3. The manufacturer's User Instructions for size selection shall be followed to determine consistency with NIOSH Panel cells for facial measurements.
  - 4.4. Each respirator shall be probed for purposes of measuring concentrations of aerosol inside the facepiece. The optimum sampling probe position is approximately 1/4 inch from the skin at the point of quadrilateral symmetry of the mouth and nose, i.e. midway between the nose and upper lip. The exact position(s) of the sample probe(s) will depend upon the design of the mask being evaluated.
  - 4.5. A short length of PortaCount tubing will then connect the sample probe(s) in the mask to the aerosol detector unit. The method in which the sampling probe(s) is used shall not interfere with respirator performance and shall minimize sampling biases.
5. Procedure
- 5.1. Turn on, calibrate equipment and measure the particle count in the ambient atmosphere. The ambient atmosphere must contain at between 1,500 and 3,000 particles per cc of air.
  - 5.2. Each subject shall be allowed to read the test activity sheet and then complete the daily pre-test questionnaire and be dismissed for cause. See appendix C & D.
  - 5.3. Each test subject shall not be permitted to eat or smoke for at least 30 minutes before the start of the test or between re-donnings.
  - 5.4. Each test subject shall perform an assisted donning of the respirator facepiece in accordance with the manufacturer's User's Instructions.
  - 5.5. Each test subject will be permitted time to make the appropriate adjustments to the facepiece until they are satisfied that they are wearing the facepiece in compliance with the manufacturer's User Instructions.
  - 5.6. Each test subject shall perform a user seal check in accordance with the manufacturer's User's Instructions. Any test subject not being able to successfully perform a user seal check shall be allowed to continue the test, but the fact that a seal check could not be performed shall be noted.
  - 5.7. Each test subject shall wear the respirator facepiece for approximately 5 minutes before beginning the test.

5.8. The actual TIL value shall be recorded in accordance with the PortaCount instructions, except as noted herein, for each test subject while performing the following sequence of exercises for 30 seconds each.

Normal Breathing

Deep Breathing

Turn Head Side to Side while pausing for two normal inhalations at each side

Move Head Up and Down while pausing for two normal inhalations in the head up position and in the head down position

Recite the Rainbow Passage in a normal speaking voice (see appendix C)

Reach for the Floor and Ceiling while pausing for two normal inhalations in the arms-up position and in the arms-down position

Grimace (not included in the TIL determination)

Normal Breathing



5.7 If the subject does not obtain a passing reading, the subject shall doff the respirator, wait at least 1 minute, then re-don the respirator and repeat the procedure. The procedure shall be repeated until a passing result or until three individual donnings and tests are completed.

5.8 Any test subject not receiving a pass after three tests is considered to have failed.

Note: If the subject fails and the respirator being tested has more than one size that covers the range of cells being used then the test subject can be retested using another size and the second series of tests shall be used to determine pass/fail.

5.9 All test subjects' comments shall be noted on the test data sheet.

## 6. Recording

6.1. The actual TIL values shall be recorded in accordance with the software instructions.

6.2. All respirator, test subject and test administrator identifiers shall be recorded.

## 7. Pass/Fail Criteria


7.1 Pass/failure criteria is given in 42 CRF Part 84, §84.175(i)(4) .

7.2 At least one test subject from each cell of the panel appropriate for the respirator being tested must obtain a passing result.

DRAFT

**Appendix A**

**NIOSH - NPPTL BIVARIATE TEST PANEL**

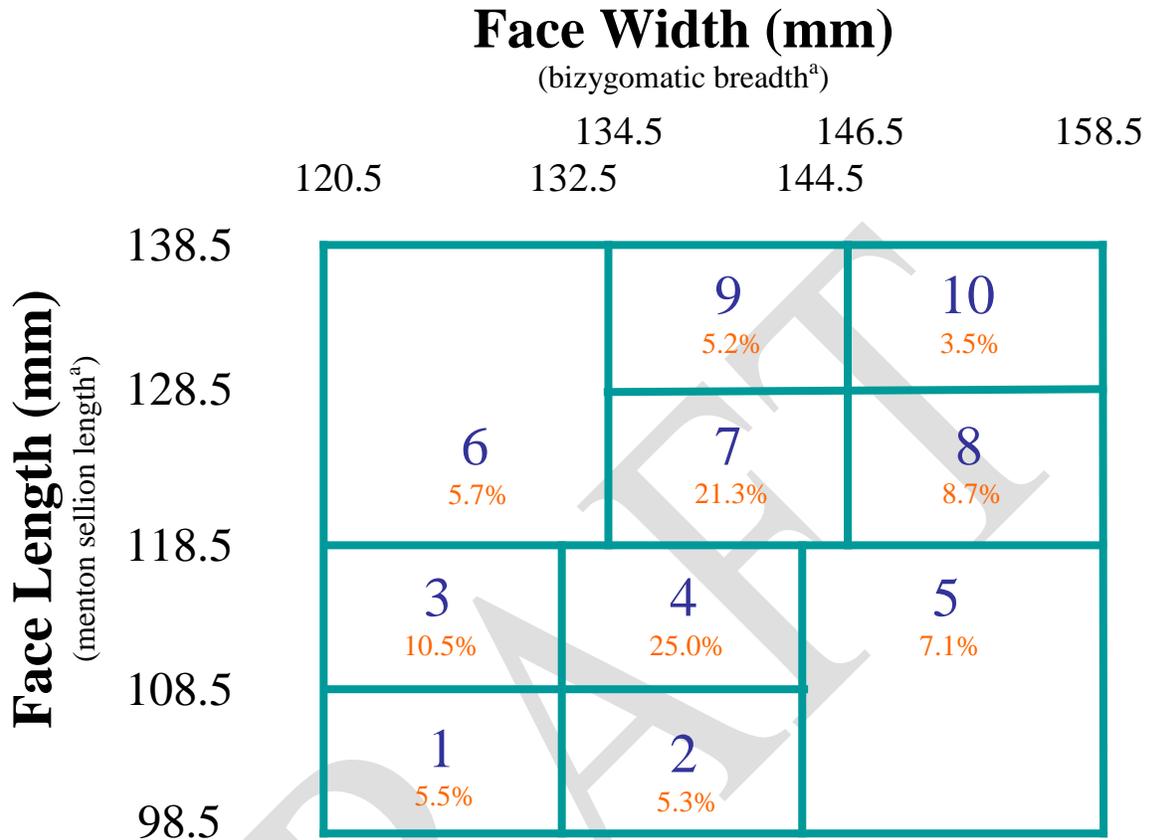

The NIOSH Bivariate Test Panel represents ≈ 98% of US respirator wearers. Cell number and percentage of the total is indicated in each cell.

**Panel population**

Test subjects within each cell have the following facial dimensions:

Cell number	Face length (mm)	Face width (mm)
1	99 - 108	121 - 132
2	99 - 108	133 - 144
3	109 - 118	121 - 132
4	109 - 118	133 - 144
5	99 - 118	145 - 158
6	119 - 138	121 - 134
7	119 - 128	135 - 146
8	119 - 128	147 - 158
9	129 - 138	135 - 146
10	129 - 138	147 - 158

<sup>a</sup>: see appendix B, Description, definition and diagram of measurements

## Panel Use

For the purpose of measuring the TIL, 35 test subjects will be chosen for a single size or multiple size respirator designed to fit the entire population, or 15 or more test subjects for a single size or multiple size respirator intended to fit one or more specific subpopulations of respirator users matching the respirator sizing as described in instructions for use accompanying the respirator.

The number of test subjects per cell is the total number of subjects multiplied by the percent of population in each applicable cell. No cell used shall have less than 2 test subjects.

### Example 1

For a test panel for a respirator designed to fit the entire population (independent of the number of sizes), the calculated number of test subjects would be:

Cell number	% of population	Number in each cell for n=35
1	5.5	$.055 * 35 \cong 2$
2	5.3	$.053 * 35 \cong 2$
3	10.5	$.105 * 35 \cong 4$
4	25.0	$.250 * 35 \cong 9$
5	7.1	$.071 * 35 \cong 2$
6	5.7	$.057 * 35 \cong 2$
7	21.3	$.213 * 35 \cong 7$
8	8.7	$.087 * 35 \cong 3$
9	5.2	$.052 * 35 \cong 2$
10	3.5	$.035 * 35 \cong 2$

### Example 2


For a test panel for a respirator designed to fit only a medium size population (cells 4, 5, 6 & 7), which represent 59.1% (25.0+7.1+5.7+21.3) of the total population, the calculated number of test subjects would be:

Cell number	% of population	Number in each cell for n=15
4	$25/.591 = 42.3$	$.423 * 15 \cong 6$
5	$7.1/.591 = 12.0$	$.120 * 15 \cong 2$
6	$5.7/.591 = 9.6$	$.096 * 15 \cong 2$
7	$21.3/.591 = 36.0$	$.360 * 15 \cong 5$

## Appendix B

### PCA Panel

The fit test panel based on Principal Component Analysis (PCA) is shown in Figure 1. This panel covers 95.2% of the male and 97.6% of the female civilian workforce. The layout of cells is different from the bivariate panel. The limit of this panel is based on an ellipse in which more than 95% of the population is included. The inner ellipse includes about one-third of the population. The rationale for the rest of the PCA configuration is to have uniform distributions for each cell. Thus, two lines were used to divide the two ellipses into four quadrants resulting in 8 cells.

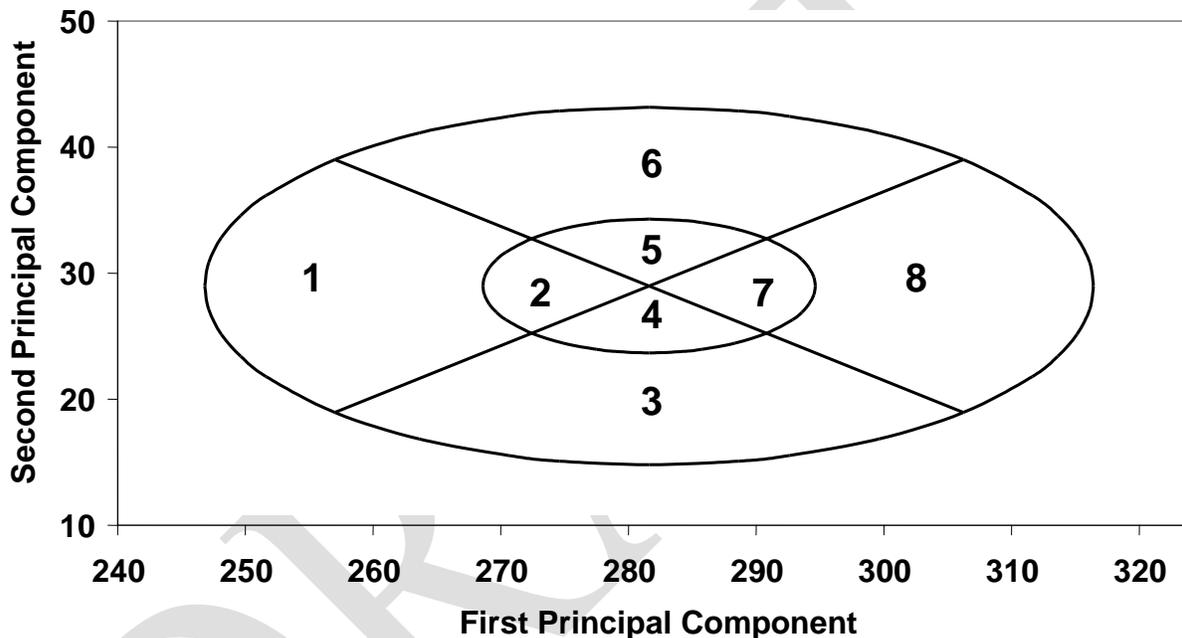

**PCA Panel Diagram**

To use the PCA panel, the 10 face dimensions need to be first measured as described in below. The first and second principal components (PC1 and PC2) are then calculated as follows.

$$\text{PC1} = 0.343264 * (\text{minimum frontal breadth}) + 0.426498 * (\text{face width}) + 0.372717 * (\text{bigonial breadth}) + 0.329648 * (\text{menton sellion length}) + 0.363474 * (\text{interpupillary distance}) + 0.372241 * (\text{head breadth}) + 0.113578 * (\text{nose protrusion}) + 0.301125 * (\text{nose breadth}) + 0.202311 * (\text{nasal root breadth}) + 0.193650 * (\text{subnasale-sellion length})$$

$$\text{PC2} = -0.152951 * (\text{minimum frontal breadth}) - 0.039087 * (\text{face width}) - 0.093279 * (\text{bigonial breadth}) + 0.359799 * (\text{menton sellion length}) - 0.173099 * (\text{interpupillary distance}) + 0.013306 * (\text{head breadth}) + 0.551842 * (\text{nose protrusion}) - 0.210833 * (\text{nose breadth}) - 0.341235 * (\text{nasal root breadth}) + 0.584261 * (\text{subnasale-sellion length})$$

The following algorithm is be used to determine which cell the subject is in.

$$x = PC1 - 281.6217618$$

$$y = PC2 - 28.9865054$$

$$\text{slope} = 5.5847930/13.6991108 = 0.4076756$$

$$a = 2.54 * 13.6991108$$

$$b = 2.54 * 5.5847930$$

$$c = 0.95 * 13.6991108$$

$$d = 0.95 * 5.5847930$$

$$r_1 = \sqrt{\frac{x^2}{a^2} + \frac{y^2}{b^2}}$$

$$r_2 = \sqrt{\frac{x^2}{c^2} + \frac{y^2}{d^2}}$$


where  $x$  and  $y$  are new coordinates for translating the origin of PC1 and PC2 from their mean values (281.6217618 for PC1 and 28.9865054 for PC2) to zero;  
slope is the slope value for the two lines dividing the ellipse into 8 cells;  
 $a$  is a constant for the length of the semimajor axis for the outer ellipse (see the illustration below);

$b$  is the constant for the length of the semiminor axis for the outer ellipse (see the illustration below);

$c$  is a constant for the length of the semimajor axis for the inner ellipse;

$d$  is the constant for the length of the semiminor axis for the inner ellipse;

$r_1$  and  $r_2$  are calculated values to determine where a particular data point or a subject is, e.g., the data point is outside the outer ellipse when  $r_1 > 1$  or on the outer ellipse when  $r_1 = 1$  or inside the outer ellipse when  $r_1 < 1$ .

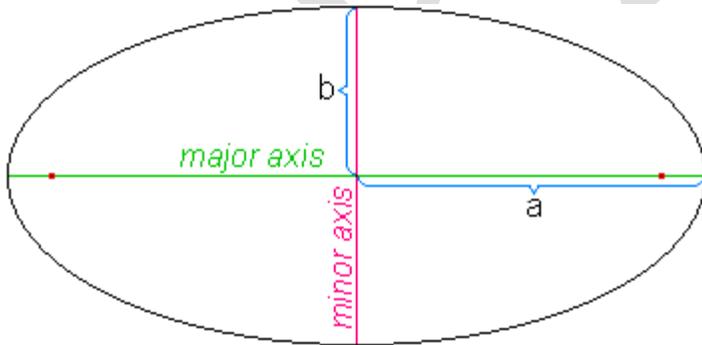

Use the  $x$ ,  $y$  and  $r_1$  values and the algorithm below to determine if the subject is in cells 1-4:

if  $x \geq 0$  and  $y \geq 0$  and  $r_1 \leq 1$  and  $\text{abs}(y)/\text{abs}(x) \leq \text{slope}$  then cell = 8

if  $x \geq 0$  and  $y < 0$  and  $r_1 \leq 1$  and  $\text{abs}(y)/\text{abs}(x) < \text{slope}$  then cell = 8

if  $x \geq 0$  and  $y < 0$  and  $r_1 \leq 1$  and  $\text{abs}(y)/\text{abs}(x) \geq \text{slope}$  then cell = 3

if  $x < 0$  and  $y < 0$  and  $r_1 \leq 1$  and  $\text{abs}(y)/\text{abs}(x) > \text{slope}$  then cell = 3

if  $x < 0$  and  $y < 0$  and  $r_1 \leq 1$  and  $\text{abs}(y)/\text{abs}(x) \leq \text{slope}$  then cell = 1

if  $x < 0$  and  $y \geq 0$  and  $r_1 \leq 1$  and  $\text{abs}(y)/\text{abs}(x) < \text{slope}$  then cell = 1

if  $x < 0$  and  $y \geq 0$  and  $r_1 \leq 1$  and  $\text{abs}(y)/\text{abs}(x) \geq \text{slope}$  then cell = 6

if  $x \geq 0$  and  $y \geq 0$  and  $r_1 \leq 1$  and  $\text{abs}(y)/\text{abs}(x) > \text{slope}$  then cell = 6

If the  $r_2$  value is less than or equal to 1, use the following algorithm to adjust the cell number:

if cell = 1 and  $r_2 \leq 1$  then cell = 7



if cell = 2 and  $r_2 \leq 1$  then cell = 4

if cell = 3 and  $r_2 \leq 1$  then cell = 2


if cell = 4 and  $r_2 \leq 1$  then cell = 5

DRAFT

Description, definition and diagram of measurements

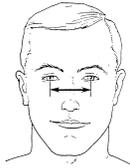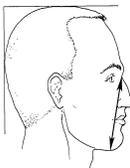Description	Definition	Diagram
Bigonial Breadth	Straight-line distance measured with a spreading caliper between the right and left gonion landmarks on the corners of the jaw.	
Bizygomatic Breadth	Maximum horizontal breadth of the face as measured with a spreading caliper between the zygomatic arches.	
Head Breadth	Maximum horizontal breadth of the head as measured with a spreading caliper above the level of the ears	
Interpupillary Distance	Distance as measured with a pupillometer at the center of the right and the center of the left pupil	
Menton-Sellion Length	Distance as measured with a sliding caliper in the midsagittal plane between the menton landmark and the sellion landmark	

Description, definition and diagram of measurements (continued)

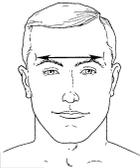Dimension	Description	Diagram
Minimum Frontal Breadth	Straight-line distance as measured with a spreading caliper between the right and left frontotemporale landmarks	
Nasal Root Breadth	Horizontal breadth of nose as measured with a sliding caliper at the sellion landmark and a depth equal to one-half the distance from the bridge of the nose to the eyes	
Nose Breadth	Straight-line distance as measured with a sliding caliper between the right and left alare landmarks	
Nose Protrusion	Straight-line distance as measured with a sliding caliper between the pronasale landmark and the subnasale landmark	
Subnasale-Sellion Length	Straight-line distance as measured with a sliding caliper between the subnasale landmark and the sellion landmark	

## Appendix c

**Activity Sheet**

Before beginning the test you will be asked to read the respirator user's instructions, put on a respirator and wear it for approximately 5 minutes to become familiar with the sensation of wearing the respirator

During the test you will be standing and will be asked to perform the following sequence of exercises for about 30 seconds each.

1. Normal Breathing
2. Deep Breathing
3. Turn Head Side to Side while pausing for two normal inhalations at each side
4. Move Head Up and Down while pausing for two normal inhalations in the head up position and in the head down position
5. Read and recite the Rainbow Passage

*"When the sunlight strikes raindrops in the air, they act like a prism and form a rainbow. The rainbow is a division of white light into many beautiful colors. These take the shape of a long round arch, with its path high above, and its two ends apparently beyond the horizon. There is, according to legend, a boiling pot of gold at one end. People look, but no one ever finds it. When a man looks for something beyond reach, his friends say he is looking for the pot of gold at the end of the rainbow."*

6. Reach for the Floor and Ceiling while pausing for two normal inhalations in the arms-up position and in the arms-down position
7. Grimace
8. Normal Breathing

You will then remove the respirator, rest for several minutes and then may perform the test sequence twice more for a series of three tests.

Appendix D

## Pre-test Questionnaire

### Total Inward Leakage (TIL) Test

Name: \_\_\_\_\_ Date: \_\_\_\_\_

Emergency Contact: \_\_\_\_\_ Phone: \_\_\_\_\_

1. Do you feel well today? .....  Yes  No
2. Have you had a cold or flu within the last two weeks? .....  Yes  No  
 If yes, how long has it been since you recovered from the cold or flu? ..... \_\_\_\_\_ Days
3. Have you eaten today? .....  Yes  No
4. Have you had at least 8 ounces of fluid in the last four hours? .....  Yes  No
5. Have you started or stopped taking any medications (or changed Doses) since your last physical exam with your physician? .....  Yes  No
6. Take a few minutes to review the activity sheet for the test that you will be performing today. Is there any reason why performing the tasks described may be unsafe? .....  Yes  No
7. Have you had an illness or injury that requires you to see a doctor or go to the hospital for treatment since your last physical exam with our physician? .....  Yes  No
8. Have you experienced any of the following conditions since your last physical exam with our physician? .....  Yes  No
 

<ul style="list-style-type: none"> <li>▪ Shortness of breath</li> <li>▪ Wheezing</li> <li>▪ Pregnancy (or possibility of)</li> <li>▪ Pain or tightness in your chest</li> <li>▪ Irregular heartbeat</li> <li>▪ High or low blood pressure</li> </ul>	<ul style="list-style-type: none"> <li>▪ Fainting or dizzy spells</li> <li>▪ Any other lung or heart problems</li> <li>▪ Unusual, severe headaches</li> <li>▪ Extremity numbness or tingling</li> <li>▪ Pain or discomfort in your legs when walking</li> <li>▪ A seizure</li> </ul>
--	--

Your Signature: \_\_\_\_\_ Date: \_\_\_\_\_

Technician's Name: \_\_\_\_\_

Yes **Do not test**                       Yes **Consult with physician before proceeding**